

## **PROTOCO<sub>2</sub>L TOUCH<sup>®</sup> Colon Insufflator**

### **INDICATIONS AND USAGE:**

The PROTOCO<sub>2</sub>L TOUCH<sup>®</sup> COLON INSUFFLATOR administers and regulates carbon dioxide as a distention media to the colon during CT Colonography (CTC or Virtual Colonoscopy) and conventional Colonoscopy.

### **IMPORTANT SAFETY INFORMATION / CONTRAINDICATIONS FOR USE:**

The PROTOCO<sub>2</sub>L TOUCH COLON INSUFFLATOR should be used only when colon insufflation is indicated, and should therefore not be used for any other treatments. It should only be used under the direct guidance of a physician experienced in colon insufflation.

This device is **contraindicated** for hysteroscopic insufflation, i.e., it must not be used for intrauterine distention.

This product should not be used in patients with known or suspected colonic perforation or toxic megacolon. It should not be used within 6 days of large forceps or “hot” biopsy, or snare polypectomy.

Do not use this product in a colostomy stoma.

Do not use this product following recent rectal surgery or low rectal anastomosis, or when proctitis or other rectal conditions such as inflammatory or neoplastic diseases are suspected.

The PROTOCO<sub>2</sub>L TOUCH Operator’s Manual may be obtained by contacting Bracco Diagnostics Inc. Professional Services Department at 800-257-5181, Option 2.

PROTOCO<sub>2</sub>L TOUCH is manufactured for Bracco Diagnostics Inc., Princeton, NJ 08540.

PROTOCO<sub>2</sub>L TOUCH is a registered trademark of E-Z-EM, Inc.

Bracco Diagnostics Inc.  
510 Carnegie Center  
Suite 300  
Princeton, NJ 08540 USA  
Phone: 609-514-2200  
Toll-Free: 1-877-272-2269 (U.S. only)  
Fax: 609-514-2446

© 2024 Bracco Diagnostics Inc. All Rights Reserved.